

EPA Reg. No. 10324-85-3838
EPA Est. No. 3838-WI-1

Readiquat™

Ready To Use • Disinfectant • Cleaner • Virucide*

SI USTED NO ENTIENDE LA ETIQUETA,
BUSQUE A ALGUIEN PARA QUE SE LA
EXPLIQUE A USTED EN DETALLE.

IF YOU DO NOT UNDERSTAND THE
LABEL, FIND SOMEONE TO EXPLAIN
IT TO YOU IN DETAIL.

KEEP OUT OF
REACH OF CHILDREN
CAUTION

See Side Panel For Additional
Precautionary Statements and First Aid.

This product is for use on hard,
non-porous surfaces in Institutions, and
Commercial and Industrial Sites.

ACTIVE INGREDIENTS:

Alkyl (50% C ₁₄ , 40% C ₁₂ , 10% C ₁₆) dimethyl benzyl ammonium chloride	0.034%
Octyl decyl dimethyl ammonium chloride	0.026%
Didecyl dimethyl ammonium chloride	0.013%
Diocetyl dimethyl ammonium chloride	0.013%
OTHER INGREDIENTS:	99.914%
TOTAL:	100.000%



733DN

NET CONTENTS:
1 QUART
(0.95 LITERS)

 **ESSENTIAL**
INDUSTRIES

DISINFECTION PERFORMANCE: This product kills the following bacteria in 10 minutes on hard, non-porous surfaces: Burkholderia cepacia, Campylobacter jejuni, Corynebacterium ammoniagenes, Enterobacter aerogenes, Enterobacteriaceae with extended beta-lactamase resistance, Enterobacter cloacae, Enterococcus faecalis, Enterococcus faecium Vancomycin Resistant, Escherichia coli, Escherichia coli (Antibiotic resistant), Escherichia coli O157:H7, Klebsiella pneumoniae, Klebsiella pneumoniae (Antibiotic resistant), Legionella pneumophila, Listeria monocytogenes, Proteus mirabilis, Proteus vulgaris, Pseudomonas aeruginosa, Pseudomonas aeruginosa (Antibiotic resistant), Salmonella enterica, Salmonella schottmuelleri, Salmonella typhi, Serratia marcescens, Shigella dysenteriae, Shigella flexneri, Shigella sonnei, Staphylococcus aureus, Staphylococcus aureus Community Associated Methicillin Resistant (CA MRSA), Staphylococcus aureus Methicillin Resistant (MRSA), Staphylococcus epidermidis (Antibiotic Resistant), Streptococcus pyogenes, Vibrio cholerae.

VIROCIDAL* PERFORMANCE: This product kills the following viruses in 10 minutes on hard, non-porous surfaces: Herpes Simplex Virus Type 1, Herpes Simplex Virus Type 2, Human Immunodeficiency Virus Type 1† (HIV-1) (AIDS Virus), Hepatitis C Virus, Hepatitis B Virus, Avian Influenza A Virus, Human Coronavirus, Respiratory Syncytial Virus, Vaccinia Virus.

† Indicates a 2-minute contact time is required for this claim.

MILDEWSTATIC PERFORMANCE: This product controls the following mold on hard, non-porous surfaces: Aspergillus niger.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

This product is not for use on medical device surfaces.

FOR USE AS A GENERAL, HOSPITAL DISINFECTANT, VIROCIDE*:

1. Pre-clean heavily soiled areas.
2. Apply this ready to use product to disinfect hard, non-porous surfaces with a sponge, brush, cloth, by immersion, auto scrubber, mechanical spray device, hand pump trigger spray device, coarse trigger spray device. For spray applications, spray 6-8 inches from surface. Do not breathe spray.
3. Treated surfaces must remain wet for 10 minutes.
4. Wipe dry with a clean cloth or allow to air dry.
5. Change solution daily or when visibly dirty.

***KILLS HIV, HBV AND HCV ON PRE-CLEANED HARD, NON-POROUS SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of hard, non-porous surfaces/objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV AND HCV ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.

Personal Protection: Wear protective latex gloves, gowns, masks and eye protection.

Cleaning Procedure: Blood and other body fluids containing HIV-1, HBV & HCV must be thoroughly cleaned from hard, non-porous surfaces and objects before application of this product.

Disposal of Infectious Materials: Blood and other body fluids, cleaning materials and clothing must be autoclaved and disposed of according to federal, state and local regulations for infectious waste disposal.

Contact Time: Allow hard, non-porous surface to remain wet for 2 minutes to kill HIV and for 10 minutes to kill all other viruses and bacteria listed on the label.

TO CONTROL MOLD/MILDEW: Apply ready to use product to hard, non-porous surfaces which will effectively inhibit the growth of mold and mildew and their odors. Repeat treatment every seven days, or more often if new growth appears.

NON-ACID TOILET BOWL AND URINAL DISINFECTANT DIRECTIONS:

Remove heavy soil prior to disinfection.

Empty water out of toilet bowl or urinal and add 1-2 oz. of this product to exposed surfaces, including under the rim with a toilet brush/mop, cloth, or sponge, hand pump trigger spray device, coarse trigger spray device. For spray applications, spray 6-8 inches from surface. Do not breathe spray. Brush or swab thoroughly, then allow solution to stand for 10 minutes and flush.

FOR USE AS A GENERAL CLEANER: Apply ready to use product to hard, non-porous surfaces. Wipe up excess liquid with a paper towel or allow to air dry.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

PESTICIDE STORAGE: Store only in original container. Keep this product under locked storage sufficient to make it inaccessible to children or persons unfamiliar with its proper use.

PESTICIDE DISPOSAL: Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture or rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste Representative at the nearest EPA Regional Office for guidance.

CONTAINER HANDLING: Non-Refillable Container. Do not reuse or refill this container. Triple rinse container promptly after emptying. Triple rinse as follows: Fill the container 1/4 full with water and recap. Shake for 10 seconds. Drain for 10 seconds after the flow begins to drip. Follow Pesticide Disposal instructions for rinsate disposal. Repeat this procedure two more times. Then offer for recycling if available or puncture and dispose of in a sanitary landfill, or by incineration.

PRECAUTIONARY STATEMENTS

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

CAUTION. Causes moderate eye irritation. Harmful if swallowed or absorbed through the skin. Avoid contact with eyes, skin or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove contaminated clothing and wash clothing before reuse.

ENVIRONMENTAL HAZARDS

This product is toxic to fish and freshwater aquatic invertebrates.

FIRST AID

In case of emergency, call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.

IF ON SKIN: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes.

IF SWALLOWED: Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by a poison control center or doctor. Do not give anything by mouth to an unconscious person. Call a poison control center or doctor immediately for treatment advice.

Sold by: **Essential Industries, Inc.**
P.O. Box 12 • Merton, Wisconsin 53056-0012 USA • (262) 538-1122

D-17